DE

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EN

remanium® star**CoCrW bonding alloy, type 5
acc. to DIN EN ISO 9693/DIN EN ISO 22674****Composition (percentage by mass)**

Co	Cr	W	Si
60.5	28.0	9.0	1.5

Additional elements < 1 %: Mn, N, Nb

Contains no nickel, beryllium or iron

Technical data

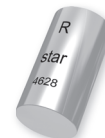
Yield strength*	R _{p0,2}	620 MPa
Tensile strength*	R _m	845 MPa
Hardness*	HV10	280
Fracture elongation*	A ₅	10.2 %
Modulus of elasticity*	E	190 GPa
Density		8.6 g/cm ³
Solidus		1320 °C/2408 °F
Liquidus		1420 °C/2588 °F
CTE (25 – 500 °C/77 – 932 °F)		14.1 x 10 ⁻⁶ K ⁻¹
Color		white
Weight per ingot		approx. 6 g

* Variations of +/- 10 % are possible depending on the batch.

Use	CoCrW bonding alloy compatible with conventional ceramic materials with appropriate CTE range.
Melting	Torch, Induction (HF), Light arc
Mirror like melt surface (HF)	Yes
Exact casting moment (HF)	As soon as the oxide layer starts to tear up
Pressure casting	250 – 450 mbar Observe instructions for use REF 989-448-00!
Welding	Possible using methods such as laser welding and TIG impulse welding
Filler material	CoCr welding wire, REF 528-215-10 (0,25 mm) REF 528-210-10 (0,35 mm) REF 528-200-10 (0,5 mm)
Presoldering	CoCrMo-Sold 1, REF 102-306-00
Flux	rema®-Flux 1, REF 102-304-00
Recommended ceramic	Ceramics with suitable CTE range such as ceraMotion® Me, Carmen® and CCS, from Dentaaurum
Cooling of ceramics in the furnace	Slow cooling, unless otherwise recommended by the ceramic manufacturer

For further information see:**remanium®**, non-precious metal alloys for crown and bridgework, instructions for use **REF 989-448-00**Available free of charge from Dentaaurum or on the internet at www.dentaaurum.com**Technical Hotline:** **+497231/803-410**

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