

Opalescence



U2
ULTRADENT
PRODUCTS, INC.

Powerful,
professional whitening
to go!





Opalescence Go whitening features ready-to-go convenience with the UltraFit™ tray for an even more adaptable and comfortable whitening experience.

- Unique tray material offers a remarkably comfortable fit and easily conforms to any patient's smile
- Molar-to-molar coverage ensures the gel comes in contact with more posterior teeth
- UltraFit tray's superior adaptation ensures the maximum amount of gel stays in contact with the teeth during whitening
- 6% hydrogen peroxide (with wear times from 60–90 minutes)

REFRIGERATE

Opalescence Go Patient Kits



| Flavor | 6% |
|--------|------|
| Mint | 4634 |
| Melon | 3592 |

Each kit contains 10 blister packs
w/1 upper/1 lower tray
1 x 20 ml (28 g) Opalescence Whitening
Toothpaste

REFRIGERATE

Opalescence Go Mini Kits



| Flavor | 6% |
|--------|------|
| Mint | 4644 |
| Melon | 3599 |

Each kit contains 4 blister packs
w/1 upper/1 lower tray

REFRIGERATE

Opalescence Go Patient Kits Case of 6



| Flavor | 6% |
|--------|------|
| Mint | 4639 |
| Melon | 3593 |

10 x Each upper/lower trays in each kit
1 x 20 ml (28 g) Opalescence Whitening
Toothpaste

REFRIGERATE

Opalescence Go Mini Kits Case of 12



| Flavor | 6% |
|--------|------|
| Mint | 4649 |
| Melon | 3600 |

4 x Each upper/lower trays in each kit

Instructions



1. Remove product from packaging.
"U" - Upper whitening tray
"L" - Lower whitening tray



2. Position upper tray on teeth.



3. Bite firmly, then suck on tray for 2 seconds.



4. Remove colored outer tray, leaving the white inner tray on teeth. Repeat the process for the lower tray.



5. After indicated wear time, remove whitening trays and brush teeth.



Learn more about the
Opalescence Teeth
Whitening family –
download the brochure!

■ Scan the QR code

Follow Us!



eu.ultradent.blog

© 2023 Ultradent Products, Inc. All rights reserved.

UPP 298-EU 01-2023

ULTRADENT.EU

U2
ULTRADENT
PRODUCTS, INC.